



★ ★ ★ **ECO** Cups

- Easy to use
- Read results at 5 minutes
- No dipping, no tipping, and no spilling
- All-in-one integrated cup
- Built-in temperature strip
(with optional adulteration strip)



ECO CUP
5-12 Panels Available

NOW
AVAILABLE

ETG ETG ETG

Ethyl Glucuronide Urine

FEN FEN FEN

Test Fentanyl Urine Test

FOR IN VITRO DIAGNOSTIC USE ONLY

AMP AMP AMP	Amphetamine Urine Test
BARB BARB BARB	Barbiturates Urine Test
BENZO BENZO	Benzodiazepine Urine Test
COC COCCOC	Cocaine Urine Test
THC THCTHC	Marijuana Urine Test MDMA
MDMA MDMA	(Ecstasy) Urine Test
BUP BUPBUP	Buprenorphine Urine Test
EDDP EDDPEDDP	EDDP Urine Test

METH.AMP	Methamphetamine Urine Test
MOR MOR MOR	Morphine Urine Test
OXY OXY OXY	Oxycodone Urine Test
OPI OPI OPI	Opiate Urine Test
PCP PCPPCP	Phencyclidine Urine Test
TCA TCA TCA	Tricyclic Antidepressant Urine Test
METHADONE	Methadone Urine Test

FOR FORENSIC USE ONLY

COT COT COT	Cotinine Urine Test
KET KET KET	Ketamine Urine Test
PPX PPX PPX	Propoxyphene Urine Test
K2 K2 K2	Synthetic Cannabinoid Urine Test

ETG ETGETG	Ethyl Glucuronide Urine Test
FEN FEN FEN	Fentanyl Urine Test
TRA TRA TRA	Tramadol Urine Test
MDPV MDPV	Methylenedioxypropylvalerone Urine Test

ECO III CUP
5-14 Panels Available




ECO II CUP
5-15 Panels Available

* "For Forensic Use Only" does not apply to any workplace testing or other non-law enforcement testing, regardless of whether or not that testing is conducted under federal agency (e.g. Department of Transportation) authority.




Tear the foil bag open, remove test cup and donor needs to obtain their own disposable gloves. Label the device with the donor's information.

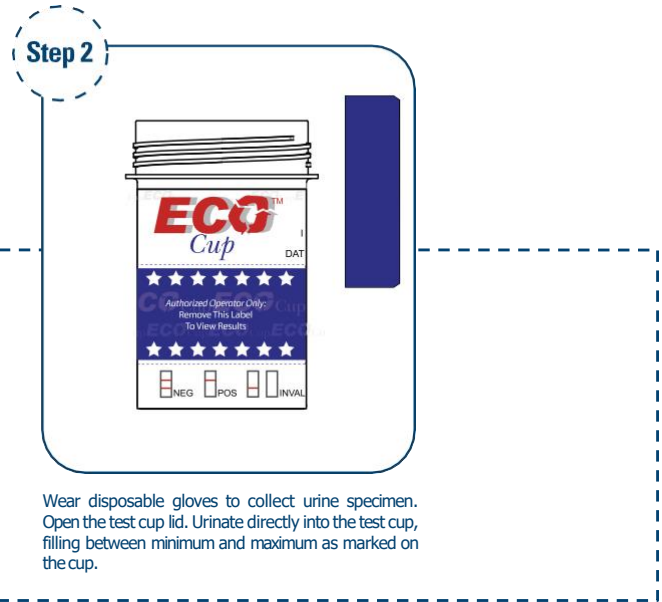

Wear disposable gloves to collect urine specimen. Open the test cup lid. Urinate directly into the test cup, filling between minimum and maximum as marked on the cup.



After the urine specimen has been collected, close the lid securely and return the cup to a collection official.

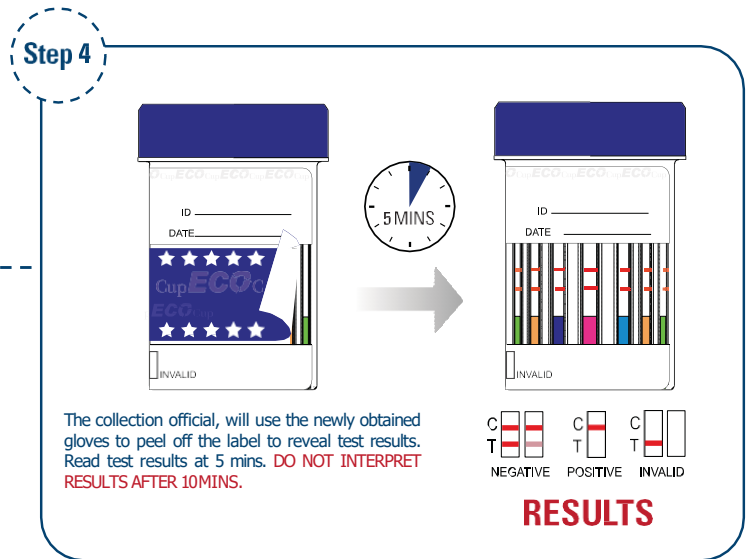

The collection official, will use the newly obtained gloves to peel off the label to reveal test results. Read test results at 5 mins. **DO NOT INTERPRET RESULTS AFTER 10MINS.**

RESULTS



Distributed By:

Sin Fronteras Medical Supplies LLC

Luis Saez

Email: info@sfmedicalsupplies.com

USA Direct Line: +1 (646) 784 9154 WhatsApp: +1 (201)

951-8809 Website: www.sfmedicalsupplies.com